

# Peak Brado

Product Code: C4500-SIB-FX, C4500-SAB-FX,  
C4500-MIA-FX, C4500-MAA-FX

## Specifications:

	Type S / M	Type S	Type M
High:		67-82 cm	78-102 cm
High folded:		67 cm	78 cm
Seat height:		52.5 cm	62.5 cm
Height adjustment of the push handles in 25 mm steps:		0-15 cm	0-24 cm
Frame High adjustable:		8 times	10 times
Lange:	66 cm		
Width:	60 cm		
Width folded:	22 cm		
Max user weight:	150 kg		
Max. Load net / bag:	5 kg		
Net weight:	7.1 kg		
Tires (puncture proof):	20 x 3 cm		

**Peak Care Equipment**

**149 Princes Highway**

**Unanderra NSW 2526**

**Tel: +612 4272 2688**

**www.peak-care.com.au**

**Distributor:**




**User Manual**

# Thank You!

Thankyou for your purchase of the Peak Brado

## Brado Rollator Parts:

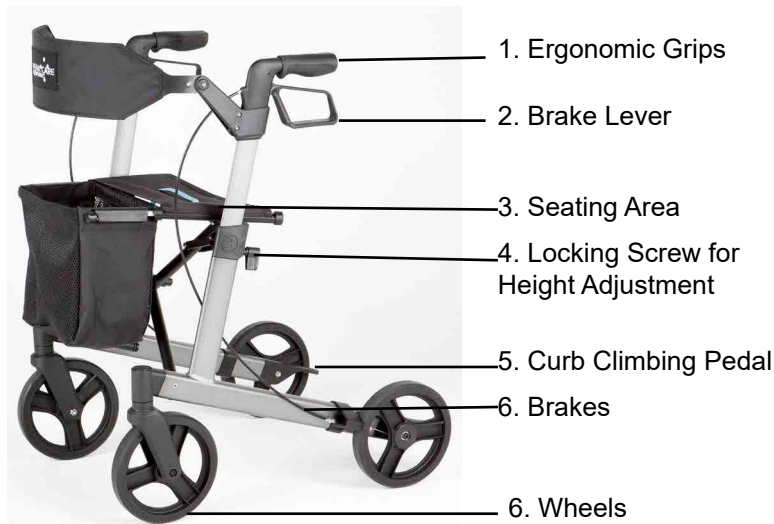

## In Your Box:

- Rollatorbrado
- Mesh pocket
- Instruction Warranty
- Card

## 12. General

- For the here described Rollator is the conformity according to the EC Directive 93/42 for medical devices explained.
- This manual is part of the delivery.
- They must be ready for the user and will continue to pass the product with the product.
- Changes due to technical advancements against over the in this Instructions versions described above, we reserve ourselves.
- Reproduction, translation and reproduction in any Form, even in part, require the written consent of the manufacturer.
- These instructions can be no changes approximation service. For current information, please check with us.

## 13. Indications and Usage

- The hinged under the seat surface mesh pocket is provided for transporting objects and goods. Please do not rely on the mesh pocket.
- The walker brado may only be used on firm, level ground. The use for other purposes, especially the use as Uphill or Transporting heavy loads is improper and dangerous!
- The walker brado serves limited mobility people with max.150 kg body weight as a walking aid and limited Seating for of transient resting.

## 9. Cleaning

- Cleaning and disinfection must in regular distances traveled by.
- Use soapy water without abrasive additives.
- The disinfection You can drove through with commercial means.
- Use no Steam jet devices and abrasive or caustic cleaning agents.
- 

## 10. Maintenance

- The walker brado is maintenance free.
- If you notice a decrease in the brakes, the wheels binding or other functional limitations, then bring the walker to be serviced by your local dealer.

## 11. Guarantee


- The warranty period is 5 years on the aluminum frame, for all remaining portions 24 months.
- Worn parts are excluded from the Guarantee.

## 1. Up / Unfold 1. Rollator

Place the rollator in the direction of travel and actuate the parking break.

Print with both hands the side tubes of the seat surface left and right down to the folding mechanism locks into place.

Only then mesh pocket, bag or basket in the throw holder provided hang.

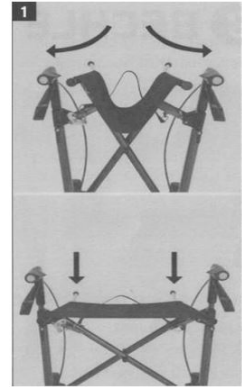


## 2. Adjust Handle Height

Solve urn set screw 5-6 turns (the screw must not be out completely) and set the desired height by pushing in or pulling the push handles.

The push handles latch into increments of 25 mm.


Turn the locking screw again. The locking screw must not be tightened between the predetermined locking positions. After tightening, the screw can be accommodated by pulling in a vertical position, thereby interfering less. between the predetermined locking positions.

After tightening, the screw can be accommodated by pulling in a vertical position, thereby interfering less.

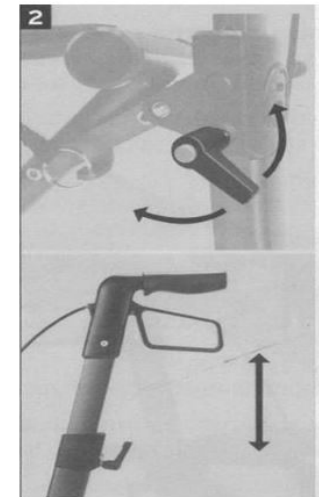

### 3. Brake

Service Brake (when pushing): Pull the lever up.  
Fasten the break (for parking): You push the lever down.

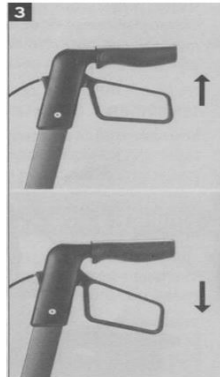

### 4. Brake Adjustment

The brake is correctly adjusted, even if the brake levers like those Bowden cables a play do not exhibit themselves and the wheels without sharpening noises to turn leave. The braking action can be set using the adjusting screw of the Bowden cable to the brake pads the rear wheels.




### 5. Climbing

Raising the steering wheels is relieved by applying pressure with the toe the prolonged so frame.



### 6. Fold Walker

Take the mesh pocket or bag on the frame.  
Pull the locking band in the center of the seating surface upward - the rollator folds together.

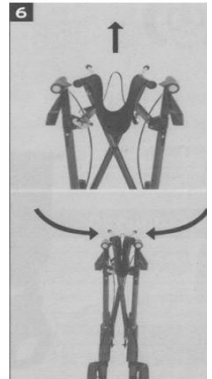

### 7. Safety

- Do not hang Pockets on the handles - Tipping!
- Use on downhill because the service brake (Pull the lever up).
- Use the rollator on firm, level ground.
- If too much lean on the handles of the walker from tipping backwards.
- When folding the walker clothing and body parts can be clamped.
- Please note that padded parts that are exposed to direct sunlight, heat up and can cause skin injuries in tubing.
- Therefore, cover these parts or protect the rollator from sunlight.
- Cleaning / maintenance / re-use

### 8. Re-use

Please keep in mind when passing the walker to the new user, the necessary technical documentation for safe handling. The rollator must be checked, cleaned and re-use by retailers for "in perfect condition" to be added.